
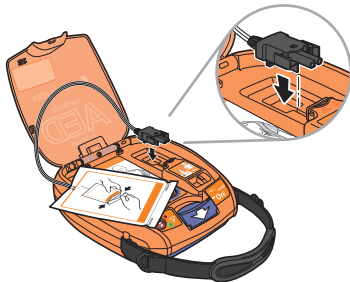


1. Connecting the Disposable Pads


1 Slide the power switch and open the lid




2 Insert the tab on the connector into the slot



3 Push the connector all the way in until it locks with a click

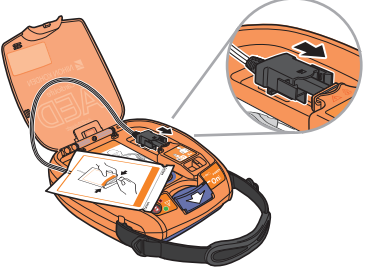


4 Store the pads on the back of the AED lid
Make sure that the illustration side of the disposable pad is facing toward you. Insert the package into the holders.




5 Run the cable through the cable guides

Run the cable through three cable guides on the back of the lid. (Indicated with arrows in the illustration)




6 Close the lid and slide the power switch



2. Installing the Battery Pack


1 Insert the battery connector

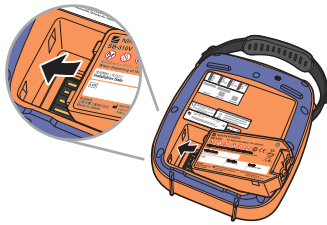

2 Firmly insert the battery pack until it clicks

Click !

Check that the battery pack is properly inserted.

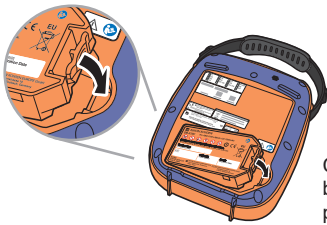

3 Check whether the status indicator is green

When you insert the battery pack into the AED, you hear a sounds. Then after about 15 seconds, the status indicator turns green. If the status indicator remains red, open the lid and follow the voice instruction.

