



1. Replace the Disposable Pads


1 First, remove the battery pack
Push in the latch and slowly remove the battery pack.

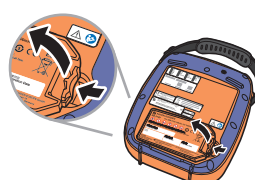


2 Slide the power switch and open the lid




3 How to remove the connector
Hold the sides of the connector and remove the connector by sliding it out.




4 Prepare the new pad




5 Insert the tab on the connector into the slot

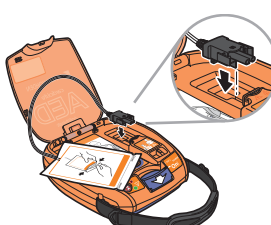


6 Push the connector all the way in until it locks with a click

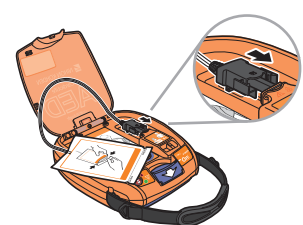

7 Store the pads on the back of the AED lid
Make sure that the illustration side of the disposable pad is facing toward you. Insert the package into the holders.




8 Run the cable through the cable guides




Run the cable through three cable guides on the back of the lid. (Indicated with arrows in the illustration)


9 Close the lid and slide the power switch



10 Insert back the battery connector

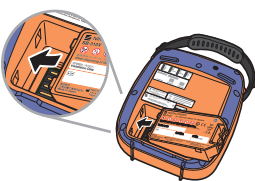


11 Firmly insert the battery pack until it clicks

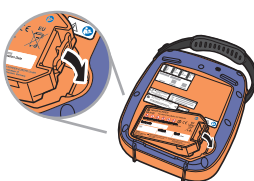

Click !
Check that the battery pack is properly inserted.


12 Check whether the status indicator is green

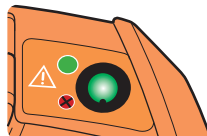


When you insert the battery pack into the AED, you hear a sounds. Then after about 15 seconds, the status indicator turns green. If the status indicator remains red, open the lid and follow the voice instruction.

2. Replace the Battery Pack


1 First, remove the old battery pack
Push in the latch and slowly remove the battery pack.

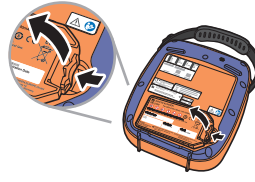


2 Prepare the new battery pack



3 Insert back the battery connector

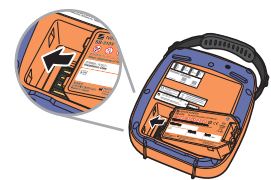


4 Firmly insert the battery pack until it clicks

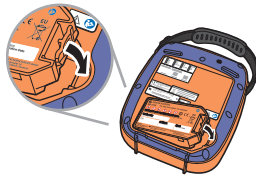

Click !
Check that the battery pack is properly inserted.

5 Check whether the status indicator is green

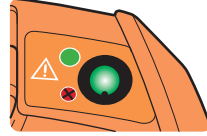

When you insert the battery pack into the AED, you hear a sounds. Then after about 15 seconds, the status indicator turns green. If the status indicator remains red, open the lid and follow the voice instruction.

Before removing the battery pack, make sure that the AED lid is closed, the AED power is off, and more than 5 seconds has passed after turning the AED power off.

Otherwise, the data might not be saved in the AED correctly.