

ActFast™

Anti Choking Trainer

Users Manual

Constructed with Durable Neoprene.

Vest front side

Transparent PVC Airway allows user to see release of obstruction.

Large easy snap buckles with adjustable straps. One size fits all

Rib Cage

User Manual window pocket

Place hands here over air bladder.

Navel aids in correct hand placement.

Vest back side



Kit Includes:

- ActFast neoprene vest - PVC bladder - Airway assembly -
- Ten foam plugs - Users manual - Storage bag
- All contents are latex-free*

For More Information:

TECSEN TECHNOLOGIES

SDN BHD (542061-K)

www.tecsen.com.my

info@tecsen.com.my

ActFast Medical LLC


for more information, www.actfastmed.com

PATENT PENDING
MADE IN CHINA
ASSEMBLED IN USA


Directions for Use

Intended for use under the direction of certified CPR Instructors.


Pull the vest over your head and cinch the waist belts snug.

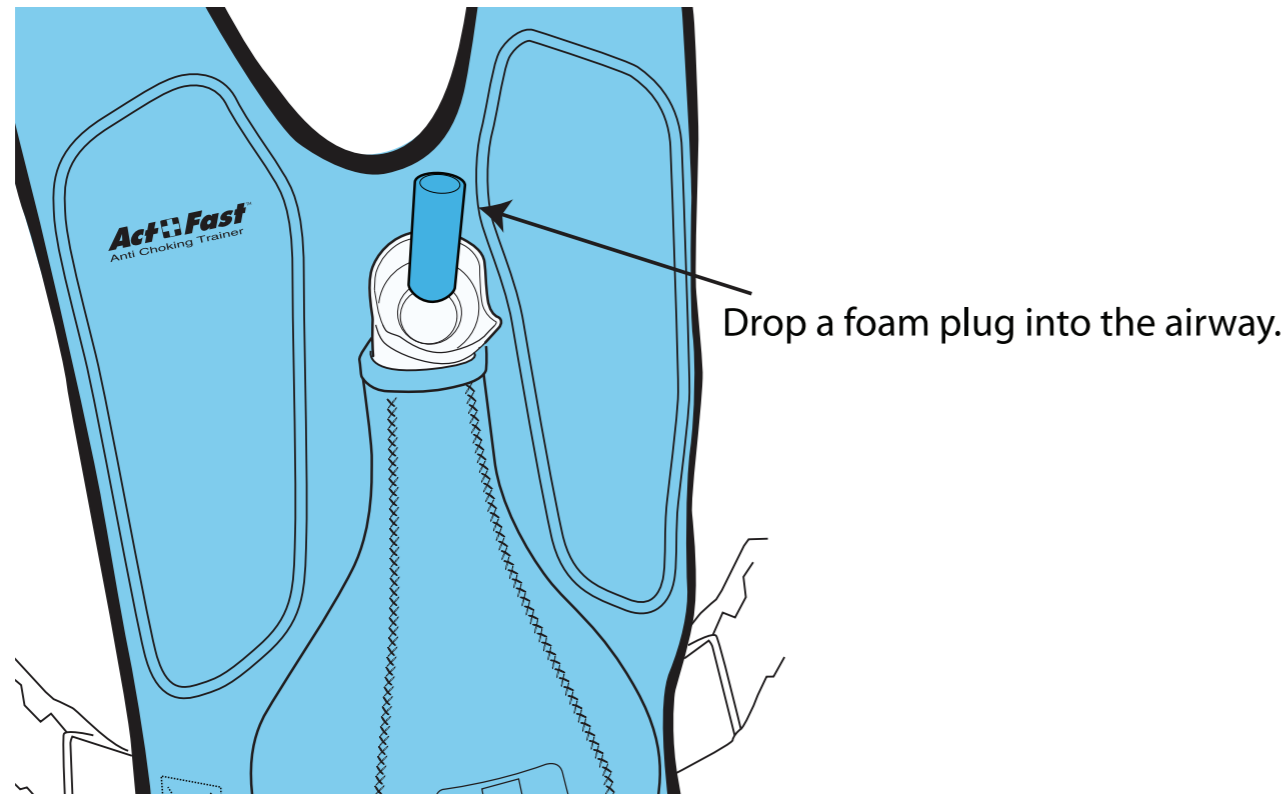

Place your hands in the correct position, between the navel and the rib cage, resting them lightly on the vest.

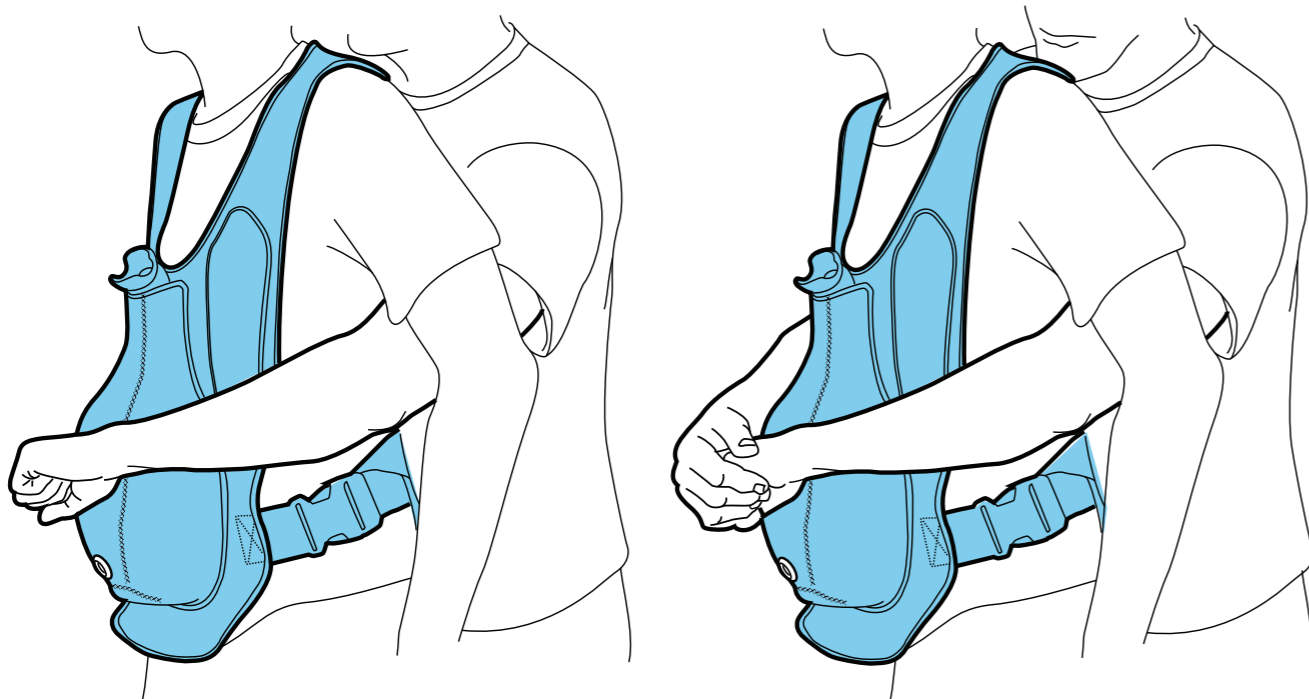

Then deliver a quick upward thrust to dislodge the obstruction. **Do not use force, it is not necessary and may cause injury.** Correct technique will result in the plug shooting into the air, providing instant feedback and teaching the concept.

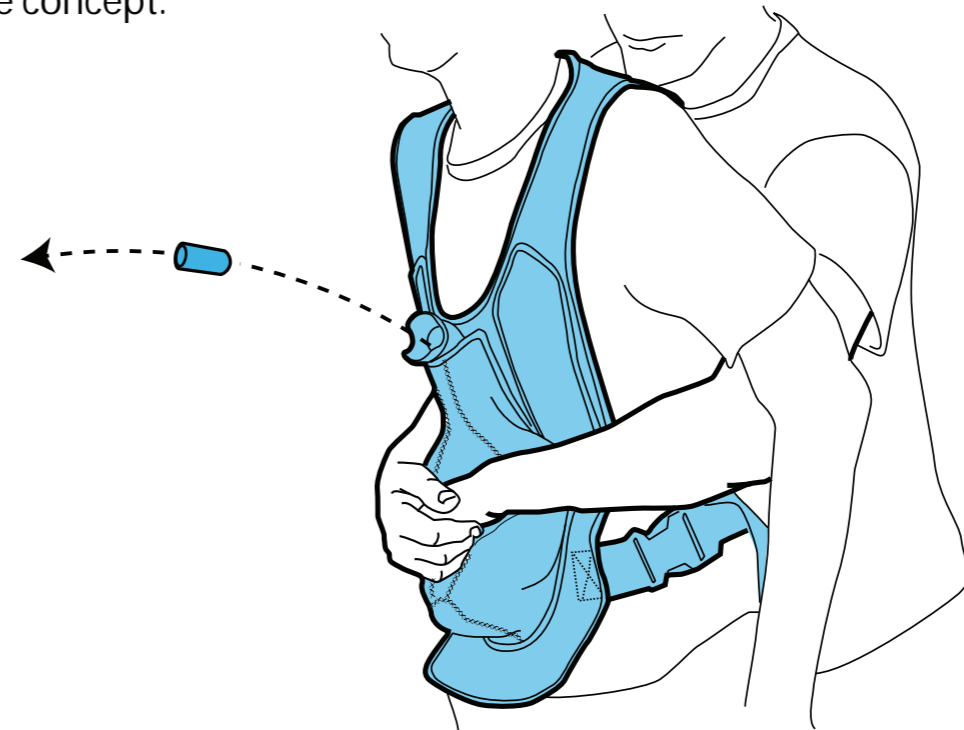


Warning: do not aim foam plug at eyes.

The training vest can be used in a variety of positions to create realistic scenarios, including: standing, sitting and laying on the ground. Also, it allows the student to practice self rescue, using a counter top or chair.

Cleaning Instructions

From the underside of the vest, release velcro straps and remove the PVC bladder and airway assembly from neoprene vest.

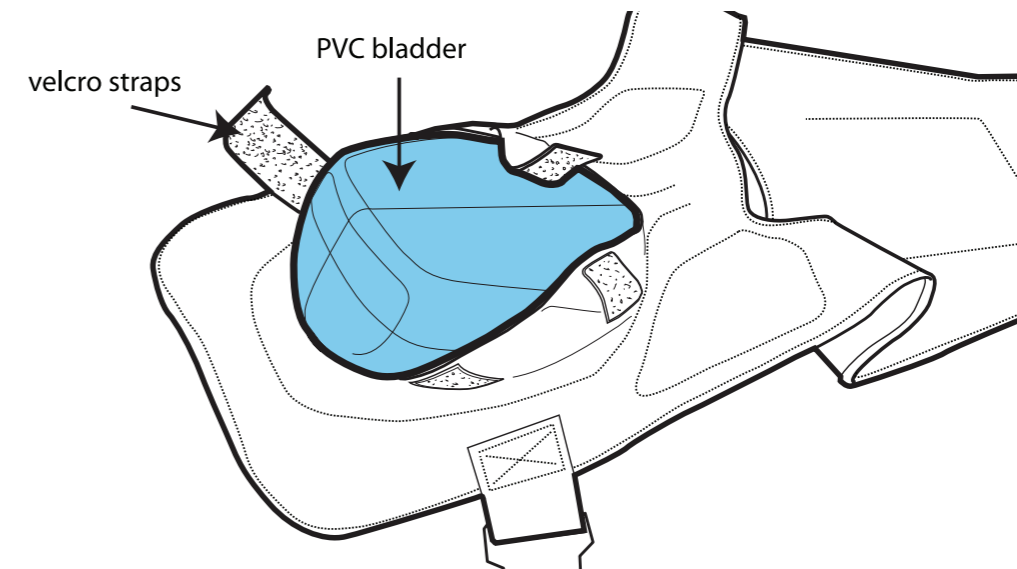

Wash vest by hand using mild detergent, hang to air dry.